Made with Love in Germany

JOYDIVISION
...my Liveskye!


Das Original.

SOFT-TAMPONS

fadenlos / stringless

your freedom!

professional

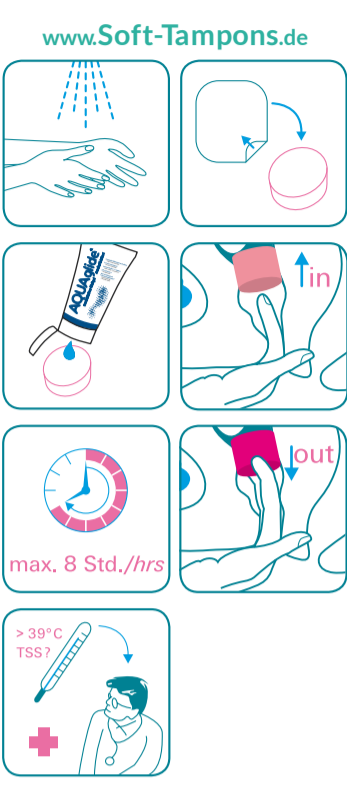www.Soft-Tampons.de

max. 8 Std./hrs

> 39°C TSS?

professional

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www.Soft-Tampons.de
EN Soft-Tampons professional
... stringless, for discreet use on a woman’s special days – especially recommended during intercourse, in the sauna, when swimming or during sport.

USER INSTRUCTIONS
Wash your hands thoroughly before inserting or removing the Original Soft-Tampons!
Insertion:

- Remove the tampon from its packaging.
- For easier insertion, we recommend that you apply a little lubricant (e.g. AQUAglide) to the dry tampon or moisten it with clean water. Squeeze firmly for maximum absorption capability before insertion.
- The tampon is easiest to insert when squatting or sitting on the toilet.
- Place your index finger on one of the tampon’s flat sides, grip the tampon by the sides using your thumb and middle finger, and squeeze it together. Use the other hand to part the labia and insert the tampon into the vagina using your index finger or middle finger until it is sitting against the cervix (see pictures).

Duration of use:
Replace the tampon frequently. Adjust the duration of use to the heaviness of your menstrual flow. During the first few days, it may be necessary to change the tampon approx. every 3 – 5 hours. We recommend you to change the tampon every 4 – 6 hours, and after 8 hours at the latest. Replacing a tampon immediately after swimming or unprotected sexual intercourse can reduce the risk of infection.

Removing the tampon:
It is best to remove the tampon using your index finger and middle finger, or thumb and index finger. A squatting position reduces the depth of the vagina from 7–8 cm to 4–5 cm, making it easier to remove the tampon. Light pressure in the lower abdomen pushes the tampon out. The use of a vaginal douche can also help.

The following always applies:
Soft-Tampons are safe for use with condoms and IUDs. Consult your physician after having an IUD freshly inserted. If

you are unable to remove a tampon by yourself, consult a gynaecologist. Don’t forget to remove the last tampon when your menstruation has ended!

PLEASE NOTE – USE EACH TAMPON ONLY ONCE!

Reusing tampons can lead to infection! Store Soft-Tampons in their original packaging and in a dry place away from light. Discolouration does not mean that the product is defective. Soft-Tampons must be disposed with the household waste, the remaining packaging in line with the country’s respective regulations concerning waste.

Important health information:

Toxic shock syndrome (TSS) is an extremely rare yet serious illness that can affect anyone. Approx. half of all TSS cases occur in menstruating women, and it has been described in combination with tampon use. Yet it can also occur when no tampon is being used! TSS can be life-threatening and must therefore be diagnosed and treated early. If, during menstruation, you should suddenly develop a fever (39 °C and above) and experience a rash, vomiting, diarrhoea, throat ache, or dizziness to the point of fainting, you should remove the tampon immediately and seek medical attention. These symptoms may be the indications of TSS. Inform your doctor that you are menstruating. Women who have suffered from TSS should consult their doctor before continuing to use Soft-Tampons.

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